



# TV • EARS®

**Dr. Recommended TV Listening Devices**

## TV Ears Link

Creates a TV Ears looped environment for T-coil hearing aid wearers as they listen to their television. Now users can keep their T-coil hearing aids in and hear the clarity of dialogue on the TV. TV Ears Link is compatible with a TV Ears 2.3MHz system.



### Step 1: Charge TV Ears Link

- Place the TV Ears Link in one of the cradles of the transmitter for 1 hour before use. The light on the left will be GREEN if placed in the front cradle or RED if placed in the back cradle. This will provide a full charge.

NOTE: the volume dial should be turned to the off position. You will hear/feel a click.

Far left light will indicate when charging



### Step 2: Ready to Wear !!

- Turn your television on
- switch your TV Ears Mobile to the Amplification & T- coil (2 beeps) mode or to the T-coil (3 beeps)only mode
- Place the TV Ears Link over your head and wear like a necklace. It should rest on your chest upper chest.
- Turn the loop on and adjust your volume level
- You can also adjust tone for your hearing



Happy TV Listening !!

Contact TV Ears Customer Service  
Toll Free: 1-800-263-4864  
Monday thru Friday, 9:00 a.m – 5:00 p.m. Eastern Time  
Visit our Website: [www.tvears.ca](http://www.tvears.ca)