



# TV EARS®

Dr. Recommended TV Listening Devices

## TV Ears Voice Cord Instructions

### Contents:

Power Microphone



Voice Cord



Extender Cord



©2007TV Ears Inc.



### Instructions:



1. Insert Power Microphone plug into the Voice Cord jack.



2. Insert voice cord plug into the "Accessory" jack on your headset.



3. Velcro the Power Microphone to the Voice Cord by removing the thin protective covering on the Power Microphone.



**Add more length to your audio cord using your Extender Cord.**

Simply connect the Extender Cord between the Power Microphone and Voice Cord.

### Battery Replacement Instructions:

The Voice Cord requires one (1) AA battery.

1. Remove screw with small Phillips from back of black box.



2. Replace the AA battery putting it into the spot marked **UM 3x2** in the battery pack.



TV Ears Canada  
1221 Barton St.  
Stoney Creek On. L8E 5G9  
Phone: (800) 263-4864 Fax: (905) 643-0001  
[www.tvears.ca](http://www.tvears.ca)

If you need further assistance,  
please contact the place of purchase.