



## **Electronic Hearing Aid Dryer**

Designed to remove moisture from your hearing aids and earphones and help to maintain peak performance.

### **Operating Instructions**

#### **In the Evening:**

1. Push the switch on the back of the Dri-Dock to the ON position. The green light indicator should illuminate.
2. Place your hearing aid(s) inside the Dri-Dock with the battery compartment of the hearing aid left open. Place the earmold with the canal aimed down. This is the same for both in-the-ear and behind-the-ear hearing aids.
3. Close the lid of the Dri-Dock. The Dri-Dock will remove moisture from your hearing aid(s) while you sleep.

#### **In the Morning:**

1. Turn the switch on the back of the Dri-Dock to the OFF position. Open the lid and remove your hearing aid(s). Allow them to cool to room temperature.
  2. Wipe the hearing aid(s) with a dry tissue.
3. Insert hearing aid(s) and enjoy another day of better hearing.

Should you experience any difficulty with your hearing aids, contact your hearing healthcare professional immediately.

#### **Limited Warranty:**

The warranty for the TV Ears Dri-Dock Electronic Hearing Aid Dryer is limited to defects in material and workmanship for 1 year from the date of purchase. Warranty is voided should any repair be attempted by anyone other than TV Ears, Inc.

For service, contact TV Ears Canada. toll free at 1-800-263-4864

Monday - Friday 9AM-5PM EST